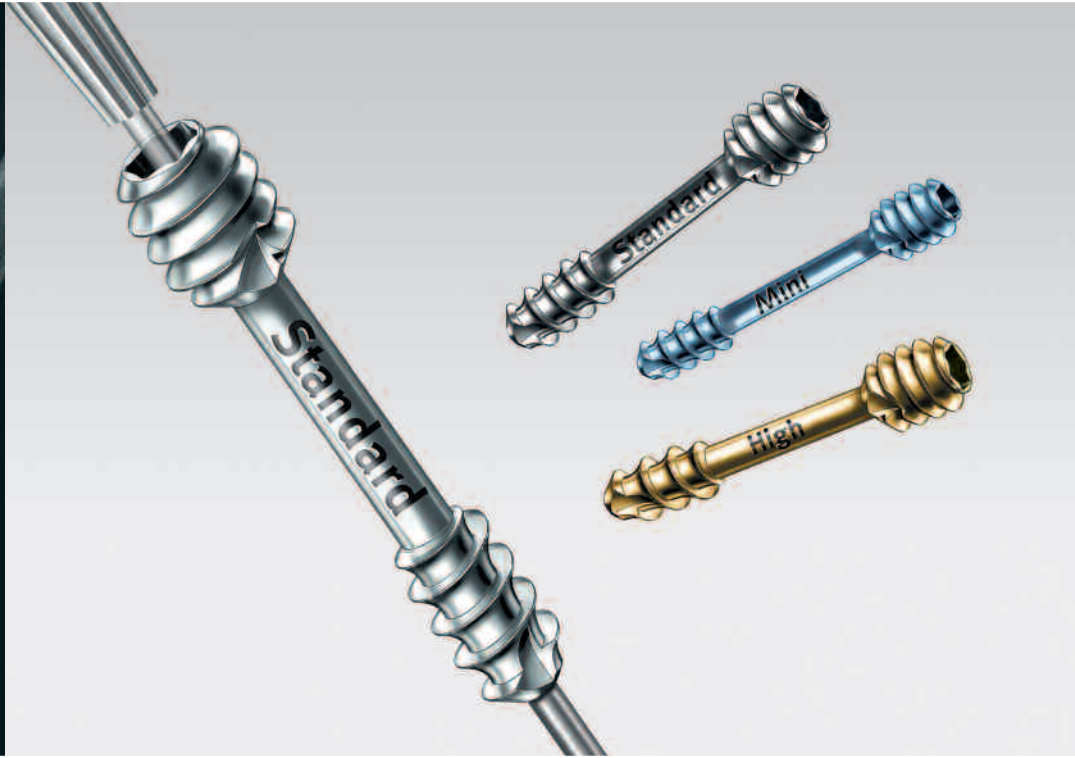
# Hand Surgery

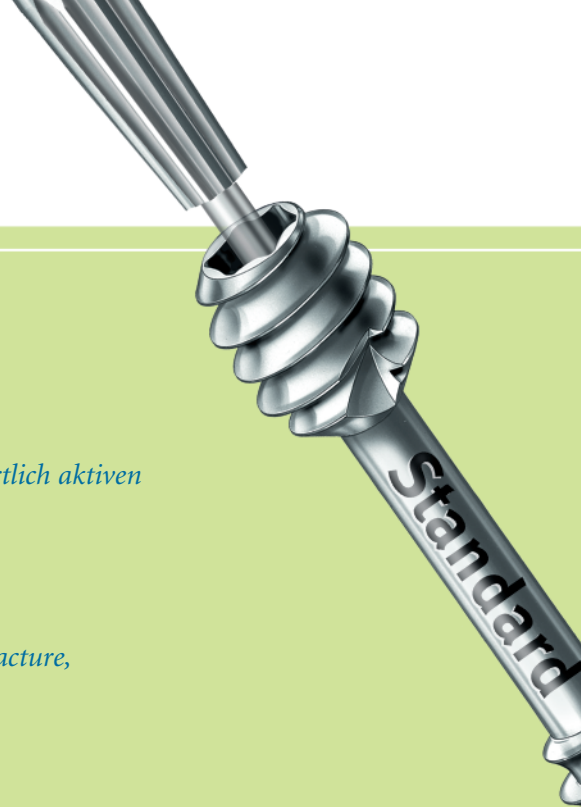
*HBS*

HEADLESS BONE SCREW SYSTEM

# HBS – Headless Bone Screw

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*Amongst all carpal bones the scaphoid is the most common one to sustain a fracture, especially as far as young and active people are concerned.*

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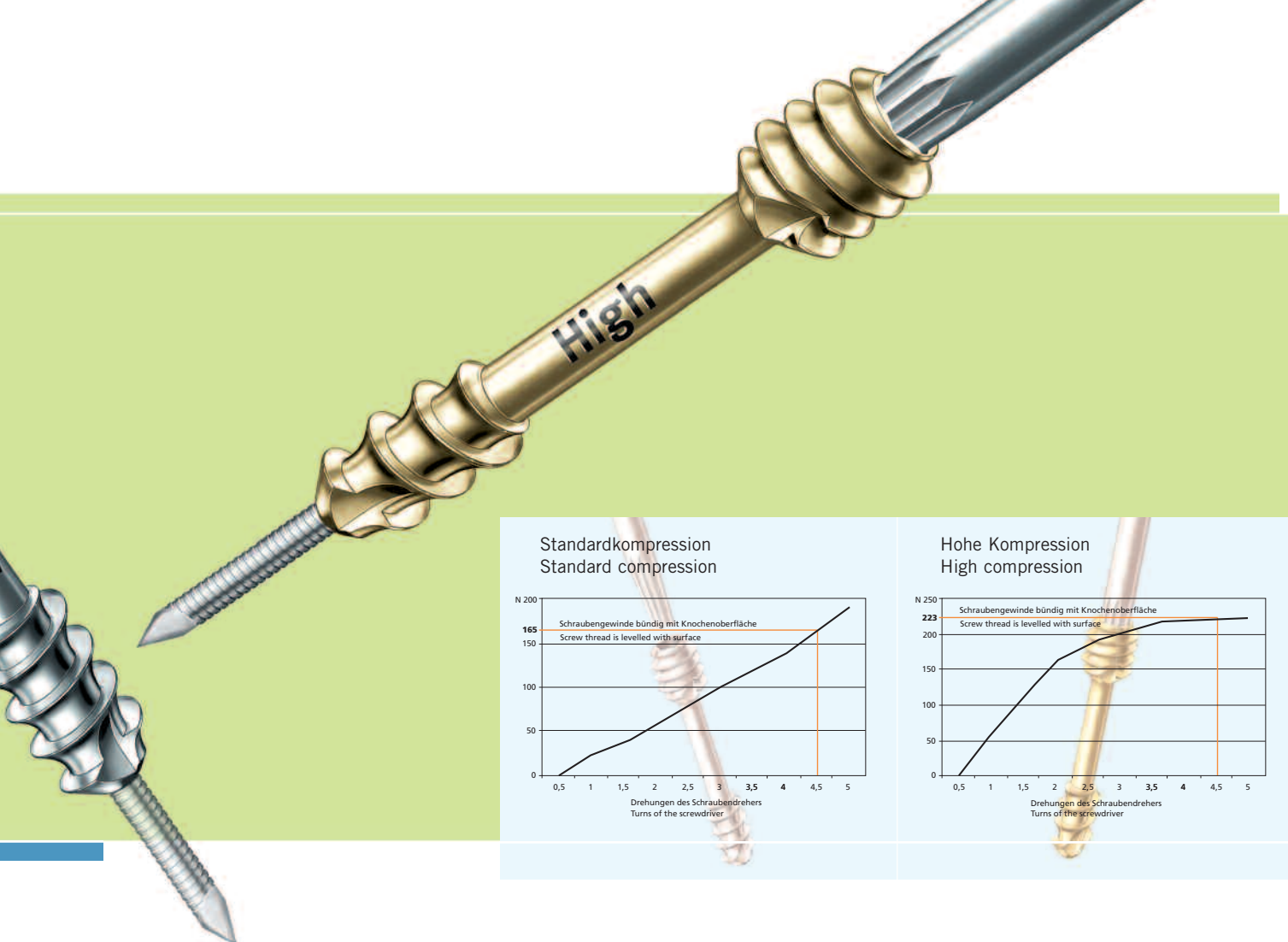
## Introduction

For the majority of scaphoid fractures immobilisation in plaster has been the most common method of treatment for a long time. This approach however means a substantial restriction as well as long periods of inability to work for predominantly young and active people. Increasing expectations of the patients require better treatment that enables early mobilisation and fast regaining of normal function. With its non-protrusive cannulated screw from the successfully approved HBS-system Gebrueder Martin offers excellent primary stability for the treatment of intra-articular fractures or those adjacent to a joint.

The System offers the surgeon two screw types, which develop different compression forces (standard or high compression). Accurate positioning of the HBS screw is facilitated by the use of a 1 mm guide wire eliminating the need for a jig. The introduction of special supplementary instruments to our system enables osteosyntheses with minimal invasive approach. Therefore the HBS-system offers all the advantages of internal fixation whilst avoiding the disadvantages of open surgery.


The T-Drive-screwdriver enables a safe and exact insertion of the self retaining screws. The implants range from 10-30 mm in 1 mm steps offering high precision of the osteosynthesis and also covering an extensive indication area within the field of hand surgery, as examples listed below demonstrate:

## Indications

- Scaphoid fractures
- Carpal fractures and nonunions
- Metacarpal fractures
- Distal radial fractures (articular fragments)
- Ulnar styloid fractures
- Radial head fractures
- Capitellum fractures
- Humeral head fractures
- Glenoid fractures
- Inter-carpal fusions
- Inter-phalangeal fusions
- Metatarsal osteotomies
- Tarsal fusions
- Malleolar fractures
- Patellar fractures
- Osteochondral fractures
- Odontoid fractures
- Mandibular fractures

## Advantages

- Cannulated screw for 1-mm guide wire
- Recessed screw (flush with bone surface)
- Two compression stages
- Self-retaining T-Drive
- Both threads are self-tapping
- Screw length in 1-mm increments
- Rotation stability
- Optional instruments for the free-hand-method
- Early mobilisation

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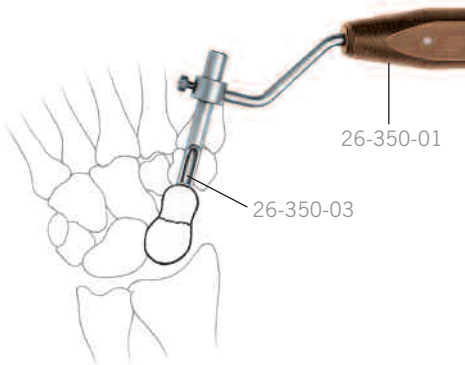
**Fig. 1:** The drill guide is assembled (26-350-03 into 26-350-01) and is then carefully positioned on the bone to be fixed.


**ATTENTION:**

It is important to carefully remove any soft tissue overlying the bone, since the points on the tip of the guide must be engaged in bone, both to prevent 'wandering' and to ensure accurate length calibration. The position and alignment of the guide may be checked on an image intensifier, as these will determine the insertion point and direction of the guide wire.

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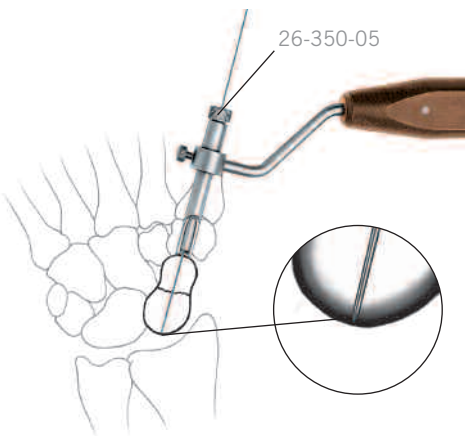
**Fig. 2:** Once the drill guide is in optimum position, the reduction sleeve (26-350-05) is introduced into the drill guide in order to control accurately the position of the guide wire (26-351-00 or 26-350-00). The guide wire is advanced slowly, under x-ray control, to ensure that it is optimally positioned perpendicular to the fracture/fusion, in the mid-axis of the bone. Its tip should enter, but not penetrate the opposite cortex of the bone. The optional tip-threaded guide wire (26-350-00) may be used to reduce the risk of the wire backing out during subsequent stages of the procedure.

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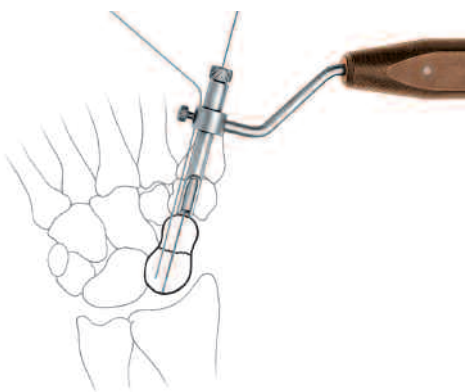**Fig. 3:** A 1 mm fixation wire (26-351-00) is now passed through one of the six peripheral holes of the collar of the guiding handle (26-350-01); this pin may be advanced across the fracture/fusion site under x-ray control, in order to maintain a good reduction during insertion of the screw. When its position is judged satisfactory, the protruding end of the fixation wire should be bent or cut short, so that it does not obstruct access to the guide. At the end of the procedure it is either removed, or else cut flush to the bone surface and left in position to provide two-point fixation.

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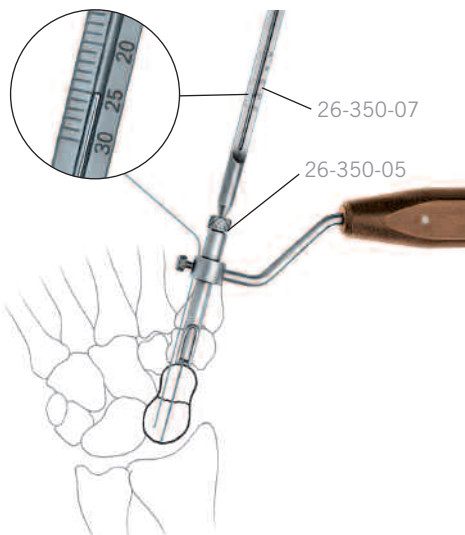**Fig. 4:** Once the fixation wire is in the correct position, the measuring sleeve (26-350-07) is passed over the protruding end of the guide wire to determine the required drilling depth.

**ATTENTION:**


To ensure accurate length measurement, the tip of the drill guide must remain firmly applied to the surface of the bone, and the measuring sleeve must be pushed over the guide wire as far as possible.

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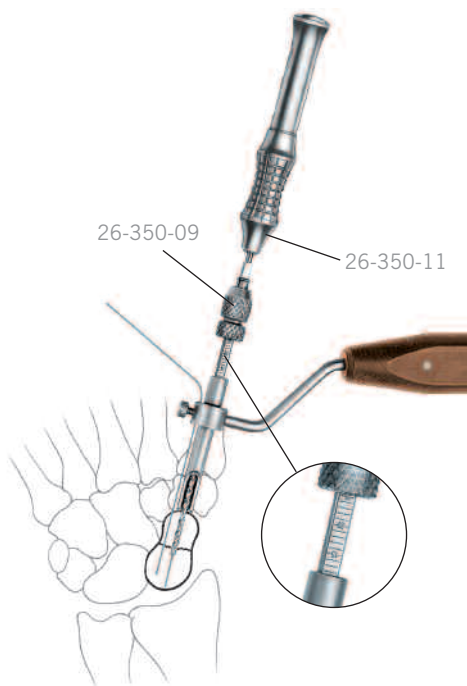
**Fig. 5:** The reduction sleeve is now removed from the drill guide and the cannulated drill (26-350-09) is passed over the guide wire, the drill stop having been set at the depth indicated by the measuring sleeve.

The drill is carefully advanced along the guide wire under x-ray control, until the drill stop reaches the end of the drill guide, indicating that the drill has reached its full depth.

**ATTENTION:**

It is recommended that the drill is withdrawn and cleaned one or more times during insertion, to prevent it from 'binding' to the guide wire; should this occur, the guide wire is likely to be withdrawn with the drill, in which case it should be repositioned (using the reduction sleeve if necessary) before moving to the next stage (Step 6).

Insertion of the guide wire and of the cannulated drill is best done using a suitable, slow-speed, small power instrument. If not available, then insertion of the cannulated drill may be done by hand, using the drill handle (26-350-11) contained in the set.

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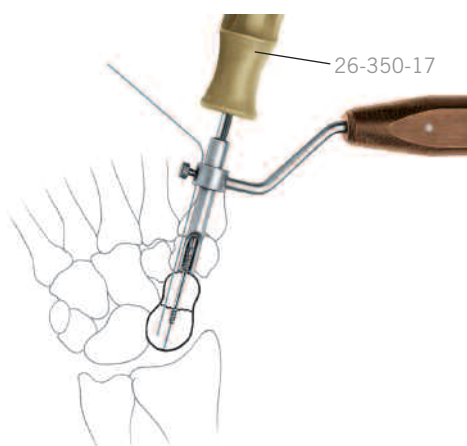
**Fig. 6:** The screwdriver (26-350-17) is used to lift a screw (26-300-xx or 26-310-xx) of correct length from the rack in the tray.

The screw is passed over the guide wire and is inserted until increased resistance is felt as the trailing thread enters the bone

**ATTENTION:**

Should increased compression be required in order to produce optimum fixation (for example in the case of a nonunion) then the high compression screw (26-310-xx) may be selected in preference to the standard screw (26-300-xx).

Due to interfragmentary compression we recommend to select a screw length which is 2 mm shorter than the adjusted drill depth.

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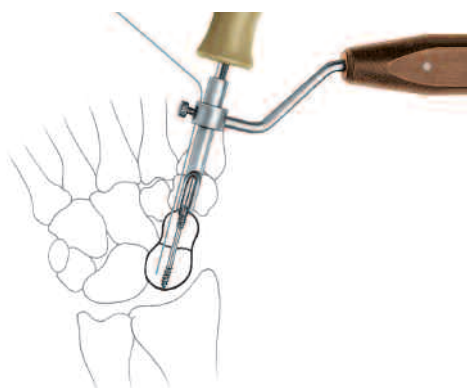
**Fig. 7:** At this stage the guide wire should be removed, the screwdriver re-inserted and the screw advanced until the screwdriver abuts on the end of the drill guide.


**ATTENTION:**

The guide wire should always be removed before the screw is fully tightened; otherwise, its tip may become bent, making it impossible to remove.

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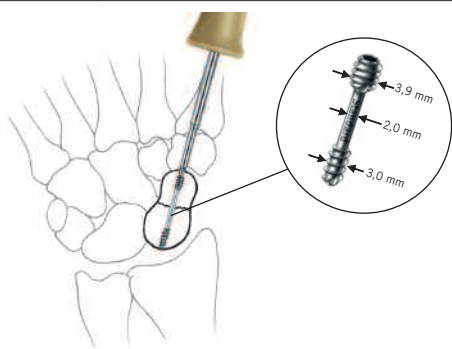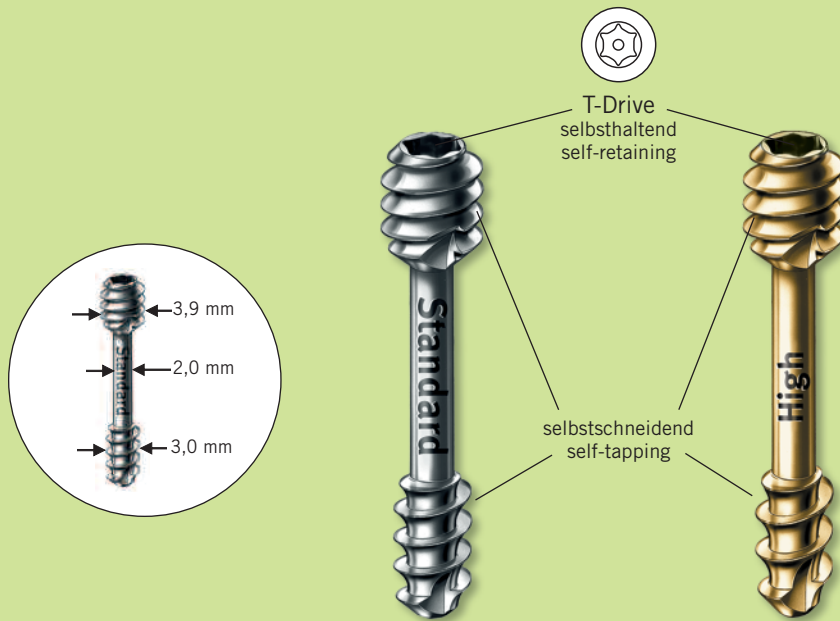
**Fig. 8:** The drill guide is now removed, and the screw is advanced a further one or two turns, in order to ensure that the trailing thread is well buried beneath the surface of the bone. If there is any difficulty in re-engaging the screwdriver, this may be facilitated by loosely repositioning the guide wire and using it to centre the end of the screwdriver in the socket of the screw.

Länge Length	Standard-Kompression Standard Compression	STERILE	R
10 mm	26-300-10	<a href="#">Ti</a>	<a href="#">1</a>
11 mm	26-300-11	<a href="#">Ti</a>	<a href="#">1</a>
12 mm	26-300-12	<a href="#">Ti</a>	<a href="#">1</a>
13 mm	26-300-13	<a href="#">Ti</a>	<a href="#">1</a>
14 mm	26-300-14	<a href="#">Ti</a>	<a href="#">1</a>
15 mm	26-300-15	<a href="#">Ti</a>	<a href="#">1</a>
16 mm	26-300-16	<a href="#">Ti</a>	<a href="#">1</a>
17 mm	26-300-17	<a href="#">Ti</a>	<a href="#">1</a>
18 mm	26-300-18	<a href="#">Ti</a>	<a href="#">1</a>
19 mm	26-300-19	<a href="#">Ti</a>	<a href="#">1</a>
20 mm	26-300-20	<a href="#">Ti</a>	<a href="#">1</a>
21 mm	26-300-21	<a href="#">Ti</a>	<a href="#">1</a>
22 mm	26-300-22	<a href="#">Ti</a>	<a href="#">1</a>
23 mm	26-300-23	<a href="#">Ti</a>	<a href="#">1</a>
24 mm	26-300-24	<a href="#">Ti</a>	<a href="#">1</a>
25 mm	26-300-25	<a href="#">Ti</a>	<a href="#">1</a>
26 mm	26-300-26	<a href="#">Ti</a>	<a href="#">1</a>
27 mm	26-300-27	<a href="#">Ti</a>	<a href="#">1</a>
28 mm	26-300-28	<a href="#">Ti</a>	<a href="#">1</a>
29 mm	26-300-29	<a href="#">Ti</a>	<a href="#">1</a>
30 mm	26-300-30	<a href="#">Ti</a>	<a href="#">1</a>


Länge Length	Hohe Kompression High Compression	STERILE	R
10 mm	26-310-10	<a href="#">Ti</a>	<a href="#">1</a>
11 mm	26-310-11	<a href="#">Ti</a>	<a href="#">1</a>
12 mm	26-310-12	<a href="#">Ti</a>	<a href="#">1</a>
13 mm	26-310-13	<a href="#">Ti</a>	<a href="#">1</a>
14 mm	26-310-14	<a href="#">Ti</a>	<a href="#">1</a>
15 mm	26-310-15	<a href="#">Ti</a>	<a href="#">1</a>
16 mm	26-310-16	<a href="#">Ti</a>	<a href="#">1</a>
17 mm	26-310-17	<a href="#">Ti</a>	<a href="#">1</a>
18 mm	26-310-18	<a href="#">Ti</a>	<a href="#">1</a>
19 mm	26-310-19	<a href="#">Ti</a>	<a href="#">1</a>
20 mm	26-310-20	<a href="#">Ti</a>	<a href="#">1</a>
21 mm	26-310-21	<a href="#">Ti</a>	<a href="#">1</a>
22 mm	26-310-22	<a href="#">Ti</a>	<a href="#">1</a>
23 mm	26-310-23	<a href="#">Ti</a>	<a href="#">1</a>
24 mm	26-310-24	<a href="#">Ti</a>	<a href="#">1</a>
25 mm	26-310-25	<a href="#">Ti</a>	<a href="#">1</a>
26 mm	26-310-26	<a href="#">Ti</a>	<a href="#">1</a>
27 mm	26-310-27	<a href="#">Ti</a>	<a href="#">1</a>
28 mm	26-310-28	<a href="#">Ti</a>	<a href="#">1</a>
29 mm	26-310-29	<a href="#">Ti</a>	<a href="#">1</a>
30 mm	26-310-30	<a href="#">Ti</a>	<a href="#">1</a>

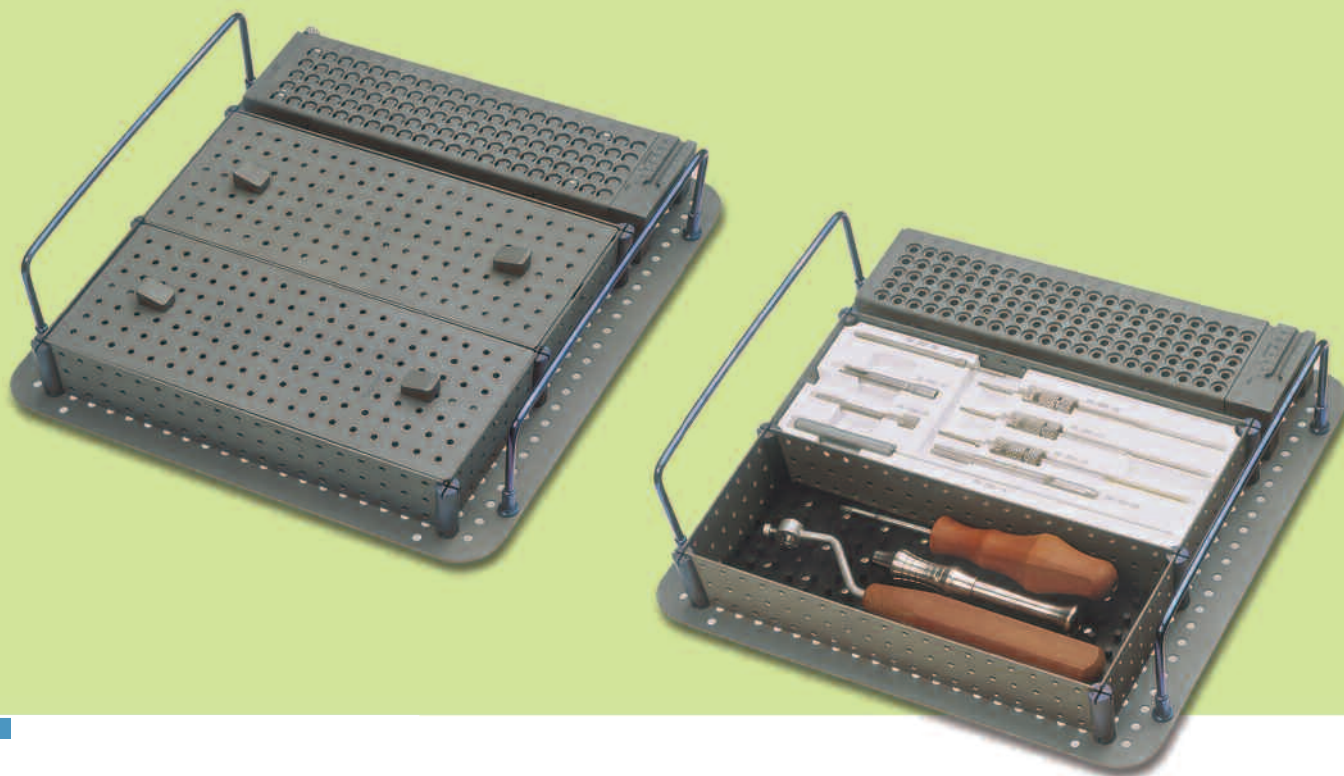26-300-00 HBS Standard-Set		26-300-00 HBS Standard Set	
<b>Implantate</b>		<b>Implants</b>	
26-300-10 - 30	Die HBS Schraube mit Standard- und	26-300-10 - 30	Set includes standard-compression
26-310-10 - 30	hoher Kompression ist in den Längen	26-310-10 - 30	and high-compression HBS screws
	10-30 mm im Set enthalten.		in the lengths 10 to 30 mm.
<b>Instrumente</b>		<b>Instruments</b>	
26-351-00	HBS Führungsdraht ohne Gewinde	26-351-00	HBS guide wire without thread
26-350-01	HBS Handgriff	26-350-01	HBS guiding handle
26-350-03	HBS Führung lang für 26-350-01	26-350-03	HBS insert for 26-350-01
26-350-05	HBS Reduzierhülse für 1-mm-Draht	26-350-05	HBS reduction sleeve
26-350-07	HBS Messhülse	26-350-07	HBS measuring sleeve
26-350-09	HBS kanülierter Bohrer	26-350-09	HBS drill bit cannulated
26-350-11	Handgriff für Bohrer 26-350-09	26-350-11	HBS handle for drill bit 26-350-09
26-350-17	HBS kanülierter Schraubendreher T8	26-350-17	HBS T-Drive screwdriver T8
26-350-19	HBS Reinigungsdraht	26-350-19	HBS cleaning wire
<b>Lagerung</b>		<b>Storage</b>	
55-732-10	Bohrdrahtspender, 16 cm	55-732-10	K-wire dispenser, 16 cm
55-910-22	HBS Lagerung	55-910-22	HBS tray
55-864-01	Codierschild mit Beschriftung, ohne Loch	55-864-01	Coding label lettered, no hole
55-864-13	Logistik-Rähmchen, blau	55-864-13	Logistics framelet, blue
55-866-13	Container 300 x 300 x 140 mm	55-866-13	Container 300 x 300 x 140 mm

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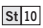26-350-02  
11,5 cm / 4 1/2"  
Gewebeschutzhülse  
Soft-tissue sleeve

1/2

26-350-06  
8 cm / 3"  
Messhülse, lang  
Measuring sleeve, long

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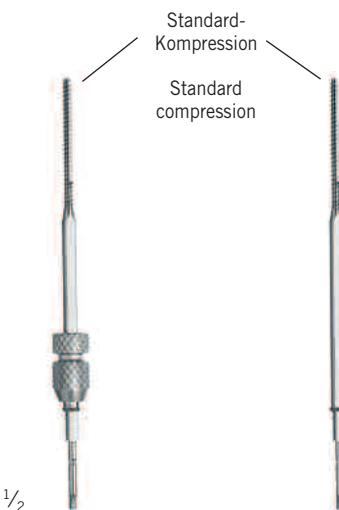
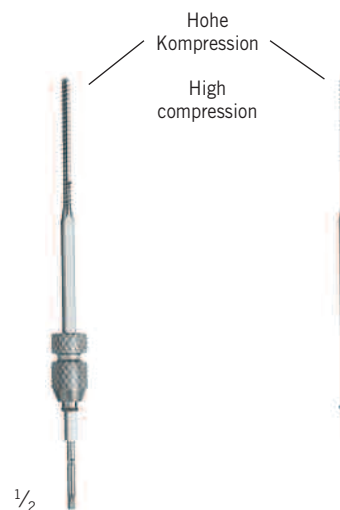
*The minituarised version of the Headless Bone Screw (HBS Mini) has a shaft diameter of only 1.5 mm, thus extending the advantages of the unique, double-threaded screw design to the fixation of extremely small bone fragments.*

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Originally developed for the fixation of small, proximal pole fractures of the scaphoid, the Mini HBS has proved useful in a range of applications including:

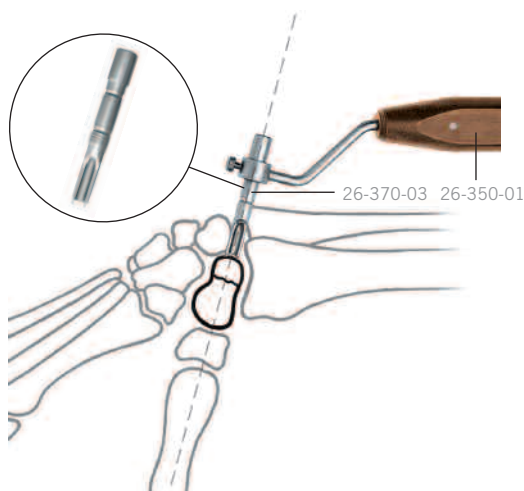
- Scaphoid fractures – proximal pole
- Scaphoid fractures – waist and distal
- In small patients
- Lunate fractures
- Capitate fractures
- Trapezial fractures
- Metacarpal and metatarsal fractures
- Phalangeal fractures (condylar)
- Radial head fractures
- Ulnar styloid fractures
- Osteo-chondral fractures
- Small joint fusions

Clinical experience has shown that this is the implant of choice for proximal pole fractures of the scaphoid: fixation is sufficiently secure to allow for active use of the wrist, thus avoiding the problems associated with prolonged immobilisation in plaster. Furthermore, the conditions for revascularisation and healing are optimised, due to the fact that the Mini HBS takes up 50 % less space within the bone than a standard implant.

Insertion of the HBS Mini is quick and simple: only a single drill hole is required, since both threads of the screw are self-tapping. The specially designed drill-guide facilitates accurate alignment of the drill, which may be power- or hand-driven. The drill is calibrated in 1 mm increments, thus ensuring that an implant of exactly the right length is inserted.

#### Advantages

- Recessed screw (flush with bone surface)
- Self-retaining T-Drive
- Both threads are self-tapping
- Screw length in 1-mm increments
- Rotation stability
- Early mobilisation

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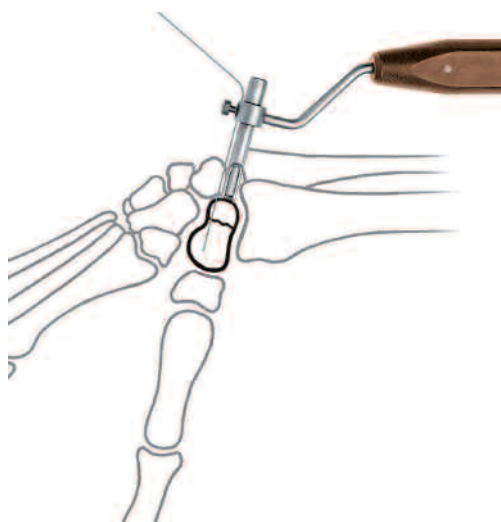
#### Step 1: Drill guide

The drill guide is assembled (26-370-03 into 26-350-01) and is then carefully positioned on the bone to be fixed. The two notches on the drill guide enable either a closer or more distant positioning of the handle on the proximal fragment.

#### ATTENTION:

It is important to carefully remove any soft tissue overlying the bone, since the points on the tip of the guide should be engaged in bone to ensure accurate length calibration.

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
#### Step 2: Temporary fixation wire (optional)

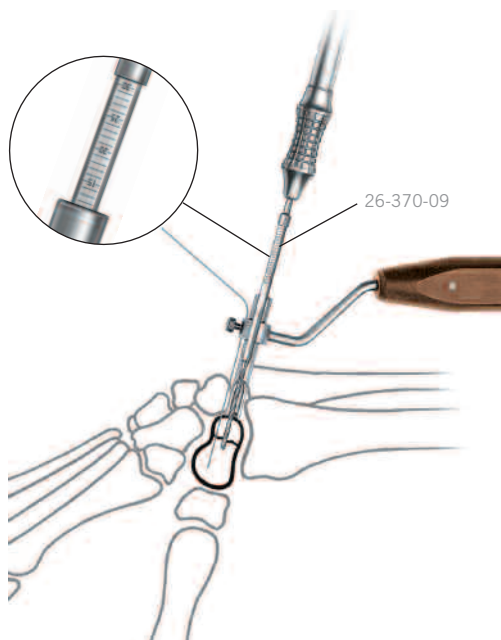A 1-mm fixation wire (26-351-00) may be passed through one of the six peripheral holes of the collar of the guiding handle (26-350-01); this pin is advanced across the fracture/fusion site under x-ray control, in order to maintain a good reduction during insertion of the screw. When its position is judged satisfactory, the protruding end of the fixation wire should be bent or cut short, so that it does not obstruct access to the guide. At the end of the procedure it is either removed, or else cut flush to the bone surface and left in position to provide 2-point fixation.

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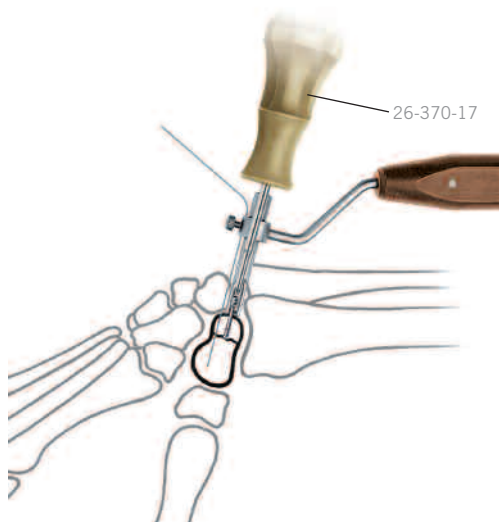
#### Step 3: Drilling


Once the fracture is accurately repositioned and the drill guide is in correct position, the drill (26-370-09) is inserted, either by hand or preferably, using a suitable power driver.

Insertion of the drill under x-ray control (image intensifier) ensures precise positioning. Once the drill has reached the required depth, the calibration is read, in order to determine the correct length of the implant.

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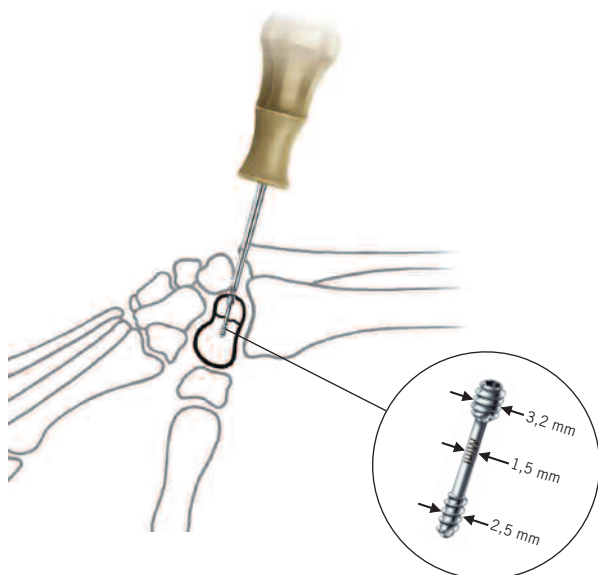**Step 4:** Insertion of the screw  
The screwdriver (26-370-17) is used to lift a screw (26-320-xx) of correct length from the rack in the tray.  
The screw is inserted until the screwdriver abuts on the end of the drill guide.

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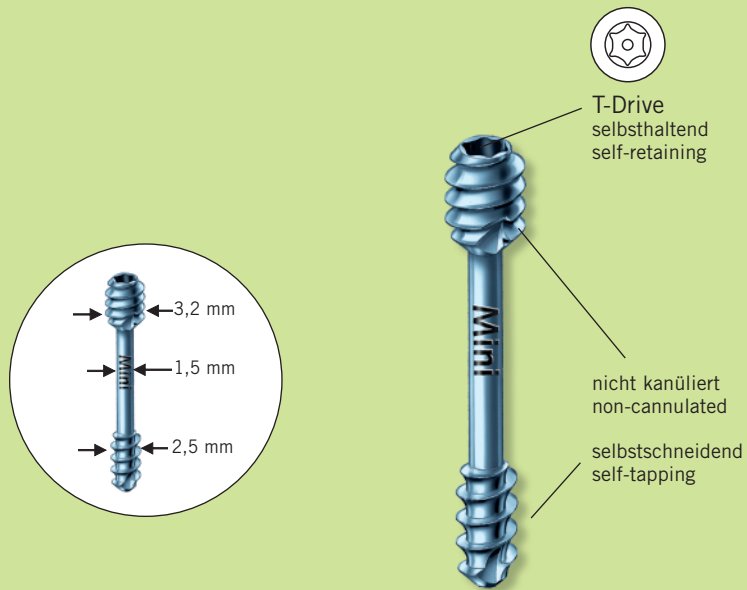
**Step 5:** Optimum compression  
The drill guide is now removed, and the screw is advanced a further one or two turns, in order to ensure that the trailing thread is well buried beneath the surface of the bone, whilst at the same time achieving optimum compression.

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Zimmer Inc., Warsaw, Indiana, 1993
**Gebrüder Martin GmbH & Co. KG**